Xblitz®

HERO

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EN

Dear Customer,

thank you for purchasing Xbitz Hero. The purchased device is a breathalyzer with an electrochemical fuel cell sensor, used to measure the alcohol content in exhaled air.

Before using the device, please read the instructions below.

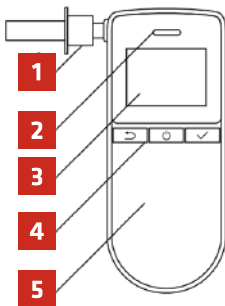
WARNING! DON'T LET YOURSELF OR ANYONE ELSE DRIVE AFTER DRINKING ALCOHOL.

01. MAIN FEATURES

1. Low power consumption and fast analysis thanks to an 8-bit microprocessor.
2. Electrochemical fuel cell sensor, which allows you to take accurate measurements of alcohol content.
3. Easy-to-read LCD display.
4. Display backlight that lets you use the device in dark places.
5. Big, easy-to-use buttons.
6. Memory of the last 10 measurements taken
7. Battery status indicator
8. Interchangeable mouthpieces ensure hygienic use of the device
9. Indicator light that shows if the measurement is above alarm level

02. DEVICE DESCRIPTION


1. Mouthpiece
2. Indicator LED
3. Display screen
4. Buttons
5. Housing

03. DISPLAY

1. Measurement value
2. Wait
3. Blow
4. Warning
5. Battery status
6. Measurement unit

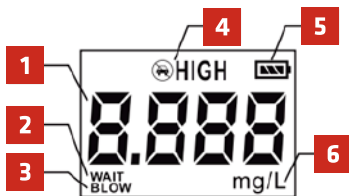EN

04. USING THE DEVICE

1. PREPARING FOR MEASUREMENT

- Remove the battery compartment cover on the back of the device. Place two 1.5V AAA batteries inside. Make sure that the battery poles are facing the correct direction. Place the battery compartment cover back in place.
 - Place a clean mouthpiece in the machine. If you are not sure where and how to put the mouthpiece, see the device diagram in the DEVICE DESCRIPTION section.
 - Press and hold the  button until you hear a short beep to turn on your device. The device display turns on and a 10-second countdown for sensor warm-up shows up on the display. Do not blow into the mouthpiece while the sensor is warming up – this will affect the accuracy of the measurement.
 - When a string of "C" appears on the display screen, take a deep breath and blow continuously into the mouthpiece until you hear a mechanical "click" sound from the device. After a few seconds, the measurement result will be displayed on the screen.
- NOTE:** If the airflow is too low or interrupted during blowing, the message "Err0" will appear on the display. Wait a few seconds before starting the test again.
- If the measurement result is below the alarm threshold, the indicator LED will turn green. If the measurement result is above the alarm threshold, the indicator LED turns red and an alarm signal will sound from the device.

f. Press the  button to turn off the result and start the next measurement.

g. In case of prolonged inactivity, the breathalyzer will turn off automatically to save batteries.

NOTE: If the test is repeated multiple times, replace the mouthpiece with a new mouthpiece every 10 measurements or clean the water/ saliva residue from inside the mouthpiece. The presence of water/ saliva in the mouthpiece can cause measurement errors.

2. VIEW SAVED MEASUREMENTS

When the device is turned on, press  to view the history of your measurements. The first recording that will be displayed on the screen is the last measurement taken. The device can record up to 10 measurements taken. Press  to return to measurement mode.

3. SETTINGS

a. Entering settings mode:

When your device is turned on, press  and then press  to enter settings mode.

b. Changing the unit of measurement:

In settings mode, press  to change the flashing digit and then press  to approve the selection.

Type 111 to change the measurement unit (mg/L by default). Tap  to change the unit and tap  to approve the selection. Press  to return to measurement mode.

c. Setting the alarm threshold:

Type 1112 to change the measurement alarm threshold. Tap  to switch to the next digit and press  to confirm the selection. Press  to return to measurement mode. By default, the alarm threshold for the device is set to 0.2‰ BAC.

NOTE: If you change the measurement unit or alarm threshold, the measurements stored in memory will be deleted.

4. TURNING OFF THE DEVICE

Hold , to turn off the device.

05. SPECIFICATION

Sensor: Electrochemical fuel cell

Measurement range: 0~2.000mg/L (0~440mg/100mL;
0~0.400%BAC; 0~4.000‰BAC;
0~4.000g/L)

Measurement resolution: 0.001mg/L

Sensor warm-up time: 10s

Operating conditions: 0 - 40°C, humidity: 20-80% RH

Storage conditions: 0- 50°C, humidity: ≤90% RH

Display: 1.5 cala LCD

Memory: Up to 10 measurements taken

Power supply: 2x AAA 1.5V battery

Dimensions: 127x 53x 22 mm

Weight: 70 g (without batteries)

Measurement error: ±0.1‰BAC for <0.8‰BAC;
±10% in the range from 0.8‰BAC
to 2‰BAC; ±30% in the range
from 2‰BAC to 0.400‰

EN

06. WARRANTY CARD

The product is covered by the 60-months warranty.
Warranty conditions can be found on the website:
<https://xblitz.pl/warranty>

Complaints should be reported via the complaint
form located on the website:
<https://reklamacje.kgktrade.pl/>

Details, contact and service address may be found at:
www.xblitz.pl

Serial number.....

The warranty is only valid for regular servicing
and calibration of the device at an authorized service
center. No less than once a year. The device from
the novelty is properly calibrated. Each breathalyzer
offered on the market requires periodic calibration,
ie every 12 months (recommended every 6 months),
the device should be configured to correctly interpret
the amount of alcohol in the exhaled air. The need for
recalibration is not a device disadvantage and is not
covered by the warranty. The device not regularly
calibrated will show incorrect results.

Manufacturer:

KGK TRADE sp. z o.o. sp. k.
Ujastek 5B,
31-752 Cracow, Poland

Made in P.R.C.

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19 / EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

07. CALIBRATION CARD

<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>

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