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USER'S MANUAL [SPIRIT]

User's manual in English language - **EN**

Read this user's manual carefully to fully enjoy the product and all of its functions.

INTRODUCTION

Xblitz Spirit is a breathalyzer equipped with an advanced semiconductor sensor with very high sensitivity and repeatability. LCS screen with backlight is simple and readable. Compact size ensures comfort of usage and storage of the device.

01. PRODUCT DESCRIPTION

A. BREATHALYZER

B. SCREEN

1. Test result
2. Unit
3. Battery indicator

Command	Test result for an exhaled air
C	gas detected, please keep on blowing
-	analyze in progress, please wait
Err	error
rP	blow error
blo	waititng for blow
bat	low battery level

EN

02. WORKING WITH THE DEVICE

1. Open the battery lid located on the back of the device. Insert two AA 1.5V batteries in correct direction (+ , -)
2. Insert mouthpiece into the slot.
3. Press & hold button for 2 seconds to turn off the device. You will hear a sound alarm and LCD screen will light up until the diode blinks with blue light and breathalyzer will start to countdown from 20.

NOTE: Do not blow into the breathalyzer before the countdown stops. Such actions may decrease the measure accuracy.

4. After finishing countdown there will be shown „blo” command on the screen. The tested person should take a deep breath and start to breath in the air slowly into the mouthpiece. There will be shown „C” command on the screen, then keep on breathing in into the mouthpiece untill you hear sound signal three times. Few seconds after that there will be a test result shown on the screen.

NOTE: During breathing in, if the air flow will not be constant the „rP” command will be shown on the screen.

5. Information diode will blink with green light when the test result is below the alarm level. Screen will display accurate test result if it will be higher than the alarm level and the diode will blink with red light. (higher than 0.2 per mil).

6. After whole process the device may be turned off by pressing and holding the button for 3 seconds.

NOTE: If there will be no operation made within the device for a longer time, breathalyzer will turn itself off automatically.

03. SAFETY

NOTE: *Never blow into the breathalyzer directly after alcohol consumption. Such action may burn the sensor and is not covered by the warranty. Wait at least 20 minutues after eating or drinking before taking a measure. Any kind of alcohol that stays in mouth or saliva may interfere with proper operation of the device. This also applies to alcohol-based mouthwash and other similar products.*

1. Fall from high altitude or violent shocks can damage the device.

2. If the battery level is low, test result may be imprecise.

3. Avoid taking a measure in strong wind conditions or in a closed room with contaminated air.

4. After long time of not using the device, first measure taken may be imprecise.

5. During taking few tests we recommend to wait for at least 1 minute to take next measure.

6. Do not blow the cigarette smoke, liquid or food into the mouthpiece.

7. After long time of non-use of the device, please use clear and soft cloth to clean the product. Do not use solvents or any other heavy substances as they may damage the device.

8. Correlaction between alcohol content in an exhaled air and alcohol concentration in blood depends on many variables, such as environment factors (air quality, wind, humidity, temperature etc.), as well as health condition of tested person.

9. You can not use the results of this device in court. The driver bears full responsibility for driving under the influence of alcohol.

10. Do not use this device in combination with ignition lock in the vehicle.

11. Test result may differ depending on many factors. Obtaining a given test result at a time does not mean that the result will be the same even a few minutes later.

12. Do not let anyone drive after alcohol consumption. Always use taxi or other vehicle with a sober driver.

13. At normal usage device demands renewed calibration every 6 months (not less than 12 months) or after 500 measures. In aim to obtain precise test results contact your distributor or service on order to calibrate the device.

04. FAQ

Error	Reason	Solution
LCD no screen	battery inserted incorrectly	Rearrange the batteries and insert then in the right direction.
	battery raddled	Replace the battery
	circuit error	Contact the technical service
No reaction while breathing in	countdown is not finished yet	Wait until the countdown stops
	circuit error	Contact the technical service
Screen error and than shutdown of the device	sensor error	Contact the technical service
I cannot turn on the device	battery raddled	Replace the battery

05. TECHNICAL SPECIFICATION

Model: Xblitz Spirit

Sensor: Semiconductor

Range: 0 ~ 0.20BAC% (0~1.00mg / L;
0~2.00 ‰ BAC; 0~2.00g/L)

Resolution: 0.01 BAC %

Heat-up time: 40s

Working

temperature: 0°C – 50°C

Storage

temperature: 0°C-50°C

Acceptable

humidity: < 93%

Screen: LCD

Power supply: 1.5V , AA batteries (x2)

Dimensions: 123mm x 44mm x 19mm

Weight: 50g (excluding batteries)

Accuracy: 0.08 BAC %

06. CALIBRATION CARD

<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>

EN

07. WARRANTY / CLAIMS

The product is covered by the 60-months warranty.
Warranty conditions can be found on the website:
<https://xblitz.pl/warranty>

Complaints should be reported via the complaint
form located on the website:
<https://reklamacje.kgktrade.pl/>

Details, contact and service address may be found at:
www.xblitz.pl

Serial number.....

The warranty is only valid for regular servicing
and calibration of the device at an authorized service
center. No less than once a year. The device from
the novelty is properly calibrated. Each breathalyzer
offered on the market requires periodic calibration,
ie every 12 months (recommended every 6 months),
the device should be configured to correctly interpret
the amount of alcohol in the exhaled air. The need for
recalibration is not a device disadvantage and is not
covered by the warranty. The device not regularly
calibrated will show incorrect results.

Importer:

KGK Trade Sp. z o.o. sp.k
os. Urocz 12
31-953 Cracow
Poland
www.xblitz.pl
Made in P.R.C.

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