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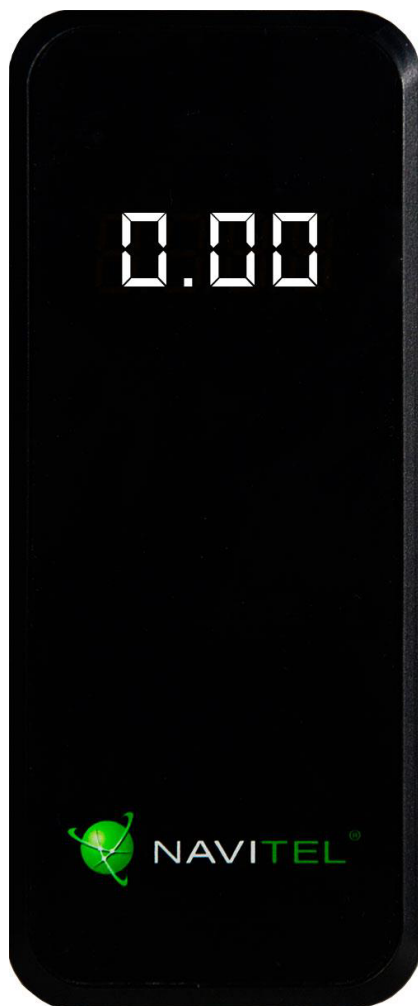
Navitel ALC 05 PRO

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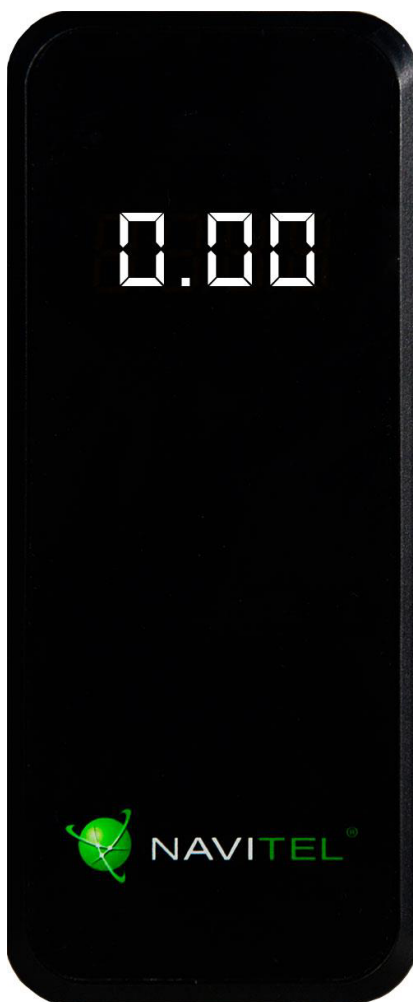
Breathalyzer User Manual

Navitel ALC 05 PRO

Thank you for choosing our product. We encourage you to familiarize yourself with its operation and user manual.

The **Navitel ALC 05 PRO** breathalyzer is a professional tool for conducting alcohol breath tests. The use of a high-quality electrochemical sensor ensures high stability and repeatability of measurements. The **Navitel ALC 05 PRO** is designed for individual customers who primarily seek high precision and affordability.

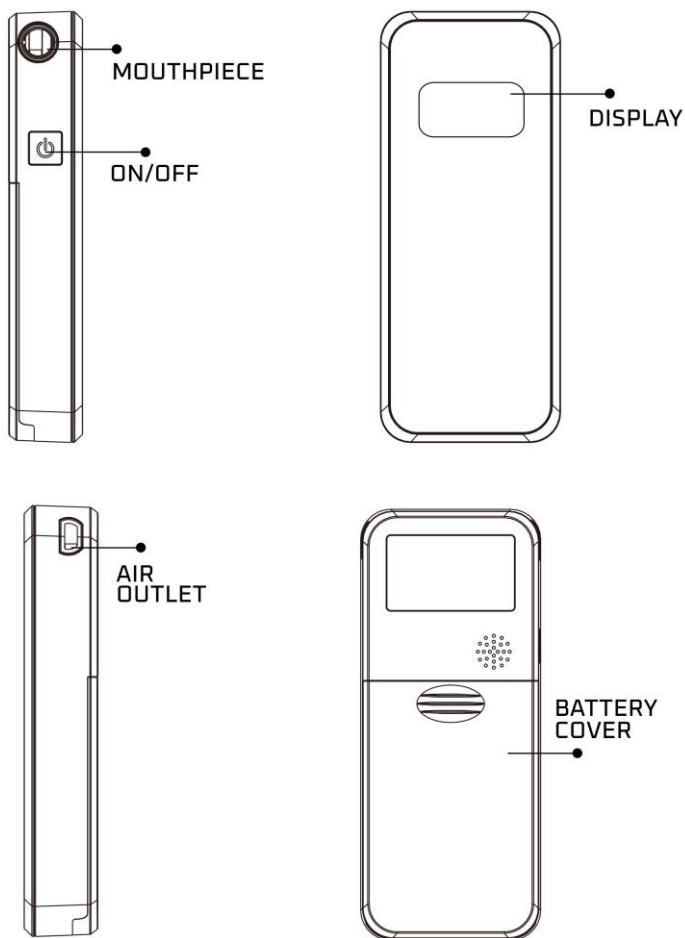
Please note: DO NOT PERFORM THE TEST IMMEDIATELY AFTER CONSUMING ALCOHOL OR SMOKING A CIGARETTE, AS YOU MAY DAMAGE THE SENSOR. Measurements should be taken at least 20 minutes after the last alcohol consumption.

POWER SUPPLY

Before use, insert the batteries provided with the purchased breathalyzer according to the markings (+ to +, - to -).

Note: If the batteries are too weak, replace them with new ones. Insufficient voltage may affect measurement results. **"Lo" will appear on the display when the batteries are running low.**

APPEARANCE

OPERATION

1. Powering on- press the **ON/OFF** button for 2 seconds to turn on the device.
2. Powering off- press and hold the **ON/OFF** button for 5 seconds to turn off the device.

Performing a Measurement with the Navitel ALC 05 PRO Breathalyzer:

1. Press and hold the **ON/OFF** button for 2 seconds to turn on the device.
2. The unit of measurement ‰ (promiles) will be displayed on the screen.
3. Press the **ON/OFF** button; the word "BLO" will appear.
4. Insert the mouthpiece, take a deep breath, and exhale. You will hear a sound signal, and "0" will flash on the display. The test result will be displayed after the clicking sound.
5. Press the **ON/OFF** button once to perform another test.

NOTE: *To perform a test immediately after a POSITIVE test, ensure that no alcohol residue remains in the breathalyzer from the previous measurement. Replace the mouthpiece with a new one and conduct a passive test (press the **ON/OFF** button when the display shows "BLO"). If the result is 0.00, proceed with another test.*

6. To turn off the breathalyzer, hold down the **ON/OFF** button for 5 seconds.

The breathalyzer has an automatic shutdown function after 1 minute of inactivity.

NOTE: *Using the same mouthpiece repeatedly (in short intervals) may result in flooding or damage to the sensor. After each measurement, we recommend replacing the mouthpiece with a new one. Residual alcohol vapors or drops may*

cause inflated measurements.

IMPORTANT

Electrochemical breathalyzers should be calibrated every 500 or more measurements, but no more than 12 months should pass from the date of purchase of the device. We recommend calibrating the Navitel ALC 05 PRO breathalyzer every 6 – 12 months. After performing 1000 tests (the device will start counting from 900 - 1000 after it is turned on), the display will show the need to calibrate "CALI". To perform the service, contact the **HUMANCARE POLSKA SP. Z O.O. Service.**

- **DO NOT PERFORM THE TEST IMMEDIATELY AFTER DRINKING ALCOHOL AND SMOKING A CIGARETTE, YOU MAY DAMAGE THE SENSOR. Measurements should be taken no sooner than 20 minutes after the last alcohol consumption.**
- When the device is not used for a long time or is in a low temperature, it will take longer to warm up
- Store the device in a dry place at room temperature
- Avoid heat or prolonged exposure to the sun
- If the difference between two consecutive readings is $>0.1\%$, the test should be repeated
- When taking a breath sample, DO NOT block the outlet
- Test results cannot be presented in court

SPECIFICATION:

Sensor type:	electrochemical
Measurement range:	0,00 - 4,00‰
Accuracy of the presented result	0.01 ‰
Measurement time:	approx. 5
Preparation time:	5 seconds
Measurement accuracy:	+/- 5 %
Recommended calibration:	every 6 – 12 months
Case:	yes
Power supply:	AA alkaline batteries 2 pcs
Dimensions:	118 x 54 x 20 mm
Weight:	129 g (without batteries)

The manufacturer, importer, distributor or seller is not responsible for damage or incorrect results resulting from improper use of the device.

Many factors (e.g. traces of various gases in the environment, smoke, other ingredients in the mouth, etc.) can affect the level of alcohol in the blood, the readings are values only as a reference point. Neither the seller nor the manufacturer accepts any liability arising from the use of the device. **HAVE YOU DRUNK? DO NOT DRIVE!**

The crossed out bin symbol indicates that this product may contain substances hazardous to the natural environment and human health as waste. Polish law prohibits, under penalty of a fine, the disposal of used parts from this device with other waste and requires them to be sent to a collection point for electrical and electronic waste. By following this type of regulation, you protect the natural environment. A recycling fee is included in the price of this product. The above information also applies to batteries if they are added to the device. Segregate waste.

MAIN DISTRIBUTOR: NAVITEL s.r.o, U Habrovky 11, 140 00 Praha 4, Czech Republic,
info@navitel.cz, www.navitel.com

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